

Suprasorb® P

PU Foam Dressing

Suprasorb® P Thoroughly absorbs exudate!

Suprasorb P absorbs exudate fast and safely in moderately exuding wounds.

The foam structure prevents adhesion to the granulation tissue and the semi-permeable PU membrane ensures the maintenance of an ideal moist wound environment.

The medium-pore, semi-permeable polyurethane (PU) foam dressing has a PU membrane

backing that repels water and bacteria. The self-adhesive version also has an adhesive border with a polyacrylic adhesive that is kind to the skin.


Indications: moderately exuding, superficial, non-infected wounds in all phases of wound healing, e.g.: decubitus, ulcers of various origins (e.g. venous, arterial), diabetic foot syndrome (DFS), skin donor and skin grafting sites, postoperative wounds, second degree burns.



Medium-pore PU foam for excellent capillary action.

The Suprasorb P dual-layer wound dressing offers numerous benefits: fast and reliable absorption of exudate, rapid swelling leads to safe contact with the wound base, maintains its shape also after absorption of fluid, easy to apply, repels bacteria and fluids, allows washing and showering, soft and comfortable to wear.

Suprasorb P is available in three versions: non-adhesive, self-adhesive and sacrum.

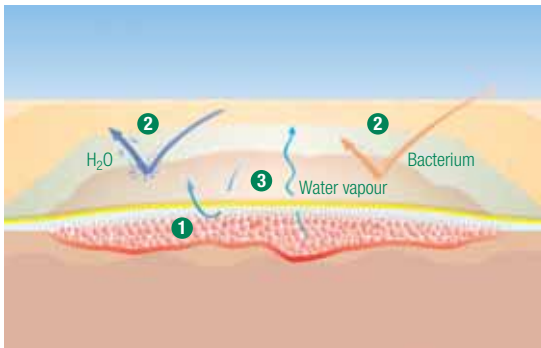

- 1 Fast absorption of exudate; rapid swelling leads to good contact with wound base.
- 2 Repels bacteria and fluids.
- 3 Surplus water vapour evaporates.

Interval between dressing changes

Depending on the degree of exudation and on the primary dressing, if used, the PU foam dressing is changed when discoloration is seen on the back and when the dressing becomes deformed to a size greater than the wound. Suprasorb P may remain in place for up to 5-7 days if the wound has thoroughly cleaned.

Suprasorb® P PU Foam Dressing, sterile

individually sealed	Size	REF	Shipping Units
self-adhesive	7.5 x 7.5 cm	20 416	10 pcs. / 100 pcs.
	10 x 10 cm	20 417	10 pcs. / 100 pcs.
	15 x 15 cm	20 418	5 pcs. / 50 pcs.
	15 x 20 cm	20 419	5 pcs. / 50 pcs.
non-adhesive	5 x 5 cm	20 405	10 pcs. / 100 pcs.
	7.5 x 7.5 cm	20 406	10 pcs. / 100 pcs.
	10 x 10 cm	20 407	10 pcs. / 100 pcs.
	15 x 15 cm	20 408	5 pcs. / 50 pcs.
	15 x 20 cm	20 409	5 pcs. / 50 pcs.
sacrum, self-adhesive	18 x 20.5 cm	20 415	5 pcs. / 50 pcs.