

Suprasorb® X + PHMB

Antimicrobial HydroBalance Wound Dressing

Suprasorb® X + PHMB Natural HydroBalance. Easy. Antimicrobial!

Suprasorb X + PHMB provides moisture regulation and additionally antimicrobial activity in lightly to moderately exuding wounds. With the unique structure of biosynthetic HydroBalance fibres plus PHMB.

In addition to the fundamental properties and advantages of Suprasorb X, Suprasorb X + PHMB contains 0.3% polyhexamethylene biguanide (PHMB) – a substance with proven antimicrobial activity. PHMB develops its antimicrobial potential not only within the dressing but also at the wound surface and effec-

tively kills microorganisms such as bacteria, fungi and yeasts.

In the context of moist wound management, it is necessary to ensure that the Suprasorb X + PHMB remains moist in the wound area when choosing a suitable secondary wound dressing (e.g. Solvaline N).


Indications: lightly to moderately exuding, superficial and deep, infected wounds in all phases of wound healing, e.g.: decubitus, ulcers of various origins (e.g. venous, arterial), diabetic foot syndrome (DFS), second degree burns, postoperative wounds, skin donor and skin grafting sites.



HydroBalance also for critically colonised and infected wounds.

Regardless of which type of wound is treated with Suprasorb X + PHMB, in addition to the properties of Suprasorb X it has the benefits of HydroBalance fibres and PHMB: wide antimicrobial spectrum, rapid effectiveness also against MRSA and VRE, good tolerance, high safety of use also in clinically manifest infections.

Suprasorb X + PHMB is available as dressing and as rope.

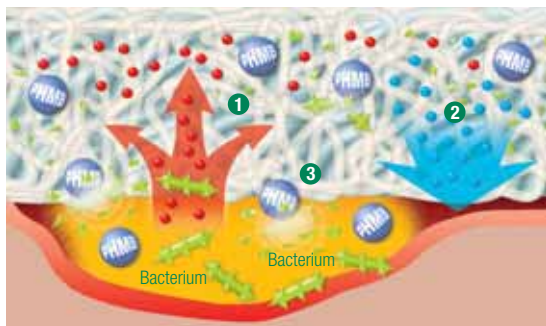

- 1 Surplus exudate from the wound is absorbed by the dressing.
- 2 Moisture from the dressing is released to dry or lightly exuding wound areas.
- 3 Killing of microorganisms by the polyhexamethylene biguanide (PHMB) that is released.

Interval between dressing changes

Depending on the degree infection, the degree of exudation and the secondary dressing used, at the latest after 5-7 days.



Suprasorb® X + PHMB Antimicrobial HydroBalance Wound Dressing, sterile

individually sealed	Size	REF	Shipping Units
Dressing	5 x 5 cm	20 540	5 pcs. / 90 pcs.
	9 x 9 cm	20 541	5 pcs. / 80 pcs.
	14 x 20 cm	20 542	5 pcs. / 65 pcs.
Rope	2 x 21 cm	20 543	5 pcs. / 130 pcs.