



Suprasorb® X

HydroBalance Wound Dressing

Suprasorb® X Natural HydroBalance. Easy. Unique!

Suprasorb X controls the moisture content of lightly to moderately exuding wounds. With the unique structure of biosynthetic HydroBalance fibres.

Suprasorb X is able to regulate the uptake (absorption) and release (rehydration) of moisture.

According to the status of the wound, surplus exudate from the wound can be absorbed by the dressing, moisture from the dressing can be released in the case of lightly exuding wounds, or the release of moisture and

absorption of exudate can take place simultaneously.

In the context of moist wound management, it is necessary to ensure that the Suprasorb X remains moist in the wound area when choosing a suitable secondary wound dressing (e.g. Suprasorb F and Suprasorb P).


Indications: lightly to moderately exuding, superficial and deep, non-infected wounds in all phases of wound healing, e.g.: decubitus, ulcers of various origins (e.g. venous, arterial), diabetic foot syndrome (DFS), second degree burns, post-operative wounds, skin donor and skin grafting sites.



Moist wound management has never been so easy!

Regardless of which type of wound is treated with Suprasorb X, there are numerous benefits: moisture regulation (HydroBalance), combats pain, wide range of application, high safety in use, usable on both sides, particularly soft and supple.

Suprasorb X is available as dressing and as rope.

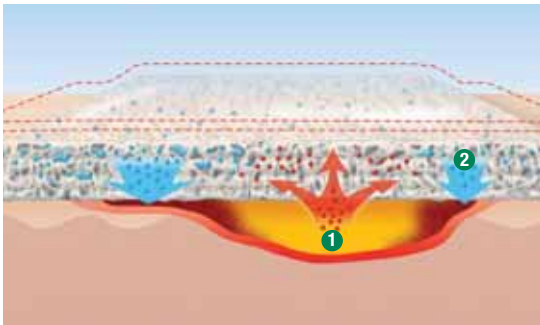

- ① Surplus exudate from the wound is absorbed by the dressing.
- ② Moisture from the dressing is released to lightly exuding wound areas.

Interval between dressing changes

Depending on the degree of exudation and the secondary dressing used, at the latest after 5-7 days.



Suprasorb® X HydroBalance Wound Dressing, sterile

| individually sealed | Size | REF | Shipping Units |
|---------------------|------------|--------|-------------------|
| Dressing | 5 x 5 cm | 20 534 | 5 pcs. / 90 pcs. |
| | 9 x 9 cm | 20 531 | 5 pcs. / 80 pcs. |
| | 14 x 20 cm | 20 532 | 5 pcs. / 65 pcs. |
| Rope | 2 x 21 cm | 20 533 | 5 pcs. / 130 pcs. |