

Cellacare® Achillo

Active support for compression and stabilisation of the area around the Achilles tendon; with an anatomically shaped silicone insert; for the therapy of acute as well as chronic, post-traumatic or postoperative irritation of the Achilles tendon, achillodynia, Achilles or sub-Achilles bursitis, peritendinitis, peritendinosis, tendinitis; postoperatively in case of rupture or partial rupture of the Achilles tendon, bursectomy, or surgery on the synovial tissue of the Achilles tendon.

Cellacare Achillo compresses the ankle joint and the area around the Achilles tendon and promotes blood circulation by friction. The special "Lymphtec" silicone insert encases the tendon canal all the way from the heel to the lower leg and actively promotes lymphatic drainage, which helps to reduce swelling and haematoma.

The shaped heel and the anatomical fit of the three-dimensional knitting fabric provide superior wearer comfort. The fabric on the back of the foot has a lesser com-



pression effect, which ensures a full range of motion and prevents pinching and constriction.

The support comes with a heel wedge, which can be placed inside the shoe for additional pain relief if necessary (another heel wedge may be used on the other foot to make up for the resulting difference in leg length and thus prevent a pelvic tilt).



Cellacare® Achillo

three-dimensional knitting, colour: blue/black; cotton/nylon fabric, insert: silicone, insert pocket: cotton.

Washable at temperatures of up to 30 °C, no chlorine bleaching, do not tumble-dry, do not iron, not suitable for dry cleaning.

Cellacare® Achillo

individually packed in a cardboard box

Size	Ankle Circumference	REF	Shipping Units
1	17 - 19 cm	24 660	1 pc. / 20 pcs.
2	19 - 21 cm	24 661	1 pc. / 20 pcs.
3	21 - 23 cm	24 662	1 pc. / 20 pcs.
4	23 - 26 cm	24 663	1 pc. / 20 pcs.
5	26 - 29 cm	24 664	1 pc. / 20 pcs.
6	29 - 32 cm	24 665	1 pc. / 20 pcs.