

Raucodrape®


A “plus” in High Performance Standard

Raucodrape® Plus unfolds its strength right where it is needed most. The field of operation is framed by 2 additional layers of reinforcing material.

With a total of 4 layers in the area of maximum stress, the material meets the most exacting demands for performance in liquid-intensive surgical procedures. Raucodrape® Plus is far above the highest standard in terms of EN 13795.

Additional safety features of Raucodrape® Plus:

- reinforced field of operation with 4 layers of material
- increased tear resistance - both dry and wet
- excellent absorption capacity
- enhanced safety in surgical procedures involving large amounts of liquids

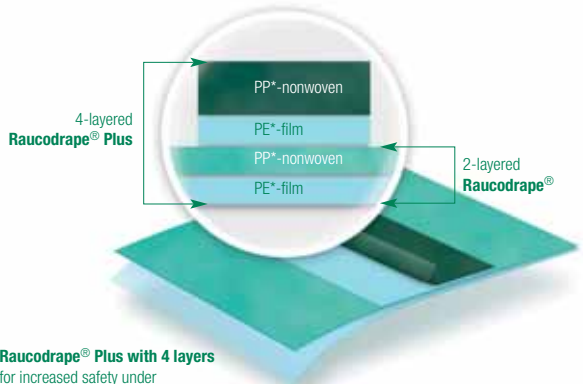

Raucodrape® Plus with 4 layers for increased safety under highest demands

* PE: polyethylene, PP: polypropylene

Tear resistance


Raucodrape Plus is strong - and right on target. Test results have confirmed its suitability or the highest demands in the OR.



all values given in Newton N; tested in accordance with ISO 9073-3
*as compared to the similar previous 3-layered version

Water absorption capacity


Raucodrape Plus absorbs 520 ml of water per m². Absorption takes place over the entire surface.

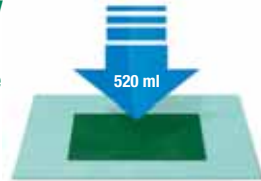

tested in accordance with ISO 9073-6

Liquid penetration

Raucodrape Plus is particularly liquid-proof. Even under a water column in excess of 150 cm, it still forms an impenetrable barrier.

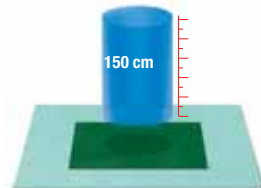

tested in accordance with EN 20811