



Suprasorb[®] **CNP** Therapy

Healing efficiently. Cost-effective.

This is your intention – and that of your team. When you apply the Controlled Negative Pressure therapy, you expect a system that is easy to use and safe. Short preparation time, fast and easy dressing changes, and economical dressing kits. In short: efficient wound management of chronic and acute wounds.



The Suprasorb® CNP (Controlled Negative Pressure) therapy is the easy to use system for Controlled Negative Pressure therapy by Lohmann & Rauscher and offers decisive benefits for successful wound healing.

The system and all its parts are easy to use and safe to operate. And the economical dressing kits provide a convincing therapy system.

Convincing therapeutic benefits:

- **the system is easy and safe to use**
reliable negative pressure pump with a high degree of user convenience, and economical kits for the application of dressings
- **therapy with a high degree of safety**
through continuous control of negative pressure and the colour coded display
- **efficient exudate management**
reliable application of negative pressure close to the wound bed
- **fast and time-saving dressing application**
sterile kits contain all the materials required for the application of dressings; they shorten preparation time and facilitate wound management
- **highly economical**
through short preparation time and reduced procedure costs
- **reliable service**
comprehensive instruction and user training
- **maximum cost transparency**
invoicing schemes geared to the customer and transparent pricing, as well as Suprasorb® CNP kits which simplify the allocation of material costs

Four easy steps for the safe application of dressings



- Step 1:** Cleanse the wound. Apply a layer of antimicrobial absorbent gauze into the wound bed.
- Step 2:** Apply the silicone drain onto the absorbent gauze. The drain can be shortened and adapted to the length and shape of the wound
- Step 3:** Fill the wound with antimicrobial absorbent gauze up to slightly above skin level.
- Step 4:** Cover the wound with a semi-permeable Suprasorb® F film dressing.

► Suprasorb® CNP Therapy

- **efficient exudate management**
- **promotes granulation**
- **supports wound bed conditioning**
- **stimulates cell proliferation**
- **reduces oedemas**
- **cleanses the wound**
- **removes bacteria and toxins, thereby reducing the bacterial load**